English

Sterile Oil for Tissue Culture

For laboratory procedures only; other uses must be qualified by the end

Product Description	REF Number	Unit Size
Oil for Tissue Culture	ART-4008P	100 mL
Oil for Tissue Culture	ART-4008-5P	500 mL
Oil for Tissue Culture	ART-4008	100 mL
Oil for Tissue Culture	ART-4008-5	500 mL

INTENDED USE

This product is intended for use in laboratory procedures that may require small volumes of tissue culture medium to be covered with a relatively inert, nonmiscible liquid component that allows diffusion of gases. Such procedures include in vitro fertilization and embryo culture and micromanipulative procedures such as ICSI and assisted hatching. Some of the proposed benefits of using small volumes of medium under oil include minimal evaporation of water from the medium, culture of gametes and embryos at increased cell densities, and the use of smaller volumes of medium containing expensive components.

COMPONENTS

Liquid petrolatum/paraffin oil/low-viscosity mineral oil.
Sterile filtered. (SAL 10⁻³)

PRECAUTIONS AND WARNINGS

Fat-soluble compounds such as steroids and fat-soluble vitamins will preferentially diffuse into the oil layer from aqueous culture medium. Spilt product can be removed with 70% alcohol and paper towel.

Do not use any container of oil that shows evidence of particulate matter or microbial contamination.

Note: Embryo is considered a general term. More precisely, SAGE™ considers the period of time initiating when a single diploid cell results from the fusion of male and female genome resulting in zygote formation with subsequent development from repeated mitotic divisions forming a solid mass or morula (typically day 4-5) and after which a fluid-filled cavity develops resulting in blastocyst formation (typically day 5-6) ending with embryo implantation that begins the end of the first week and is completed by the end of the second week post conception.

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

QUALITY ASSURANCE

One-cell MEA tested and passed with 80% or greater blastocyst. USP Endotoxin test passed with <0.1 EU/mL.

A Certificate of Analysis is available for this product.

DIRECTIONS FOR USE

It has been recommended to equilibrate the oil with 10% (v/v) of culture medium, replacing the first lot of medium with a second lot after overnight incubation in a CO₂ incubator. The container of equilibrated oil is then left in the incubator with the cap of the bottle loosely in place to allow gas equilibration.

Each laboratory should make its own determination of which medium to use for each particular procedure.

STORAGE INSTRUCTIONS AND STABILITY

Recommended storage in the dark at 2 °C–8 °C upon receipt. The product is stable until the expiration date shown on the label.
A. Remove desired volume of product using aseptic procedures.
B. Once removed, do not return any volume of product to the original container.
C. Store the sealed container in the dark at recommended 2 °C–8 °C.
D. Do not use if the product becomes discolored, cloudy or turbid, or shows any evidence of microbial contamination.

RELATED PRODUCTS

SAGE In Vitro Fertilization™ has a full line of products for the Reproductive Medicine Specialist. Please call or write for specific information or to receive a copy of our current catalog. For technical questions, or to reach our Customer Service Department, call the SAGE™ Support Line.

SAGE™ SUPPORT LINE:
In the U.S.: (800) 243-2974 International: (203) 601-9818

EXPLANATION OF SYMBOLS

- REF** Catalog Number
- LOT** Batch Number
- Use By (year, month, day)
- Temperature Limitation
- STERILE A** Aseptic Technique Sterilization Membrane Filtered (SAL 10⁻³)
- ATTENTION:** See instructions for use.
- EC REF** Authorized Representative in the European Community.
- CE** Product conforms to the Medical Device Directive 93/42/EEC
- Manufacturer
- RX ONLY** U.S. Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

SAGE In Vitro Fertilization
a CooperSurgical Company

SAGE™ In Vitro Fertilization, Inc.
a CooperSurgical Company
95 Corporate Drive
Trumbull, CT 06611 USA

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